




Short-term patient stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

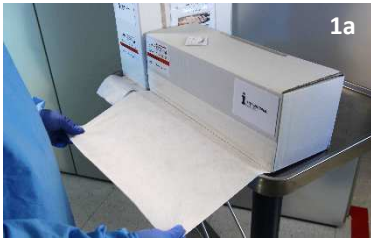

1a



1b

1

Estimate the required length of iFIX Fleece and cut.



2a



2b

2

On the opposite side of the table attach the iFIX Fleece to anchor product.

Anchor products:
iFIX Patch, iFIX Patch Adapter,
iFIX Slot Adapter



3a



3b

3

Pull iFIX Fleece tightly, up and over the patient.

Do not restrict patient's breathing.



4a



4b

4

Adhere the iFIX Fleece to the anchor product on the other side of the table.

If necessary, adjust iFIX Fleece to increase or decrease stabilization.



5a



5b

5

Repeat this procedure with a second piece of iFIX Fleece if desired.

See Instructions for Use for respective anchor products.

WARNINGS AND PRECAUTIONS

- Non-Sterile
- Single Patient Use; Do Not Reuse
- Latex Free and Hypoallergenic
- Safe for Use in MRI

INTERVENTIONAL-SYSTEMS is a registered trademark
iSYS is a registered trademark
iFIX is a registered trademark
iFIX technology patent pending
© 2017 iSYS Medizintechnik GmbH



Doc ID: QA_73_29643 - Rev. 04 / 2017-12-19 / EN-EU

iSYS REF	Product Name	Specifications
16304	iFIX Fleece 15	1 roll (200 m) per box, width 15 cm
17674	iFIX Fleece 50	1 roll (30 m) per box, width 50 cm



www.interventional-systems.com

MANUFACTURER:
iSYS Medizintechnik GmbH
Bergwerksweg 21
6370 Kitzbühel, Austria
Tel: +49 (8232) 99 41 956
info@interventional-systems.com