



Short-term patient stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.



1

Select appropriate iFIX Head Mask size for the patient.



2

Place mask over patient face ensuring eyes, nose, and mouth are exposed.
Do not restrict patient's breathing.



3

Attach iFIX Head Mask tabs to anchor product (red arrow).

Anchor products:
iFIX Patch, iFIX Patch Adapter



4

If necessary adjust iFIX Head Mask to increase or decrease stabilization.
See Instructions for Use for respective anchor products.

iSYS REF	Product Name	Specifications
26102	iFIX Head Mask XS	10 Head Masks per box
26105	iFIX Head Mask S	10 Head Masks per box
20104	iFIX Head Mask M	10 Head Masks per box
26103	iFIX Head Mask L	10 Head Masks per box
29341	iFIX Head Mask XL	10 Head Masks per box

WARNINGS AND PRECAUTIONS

- Non-Sterile
- Single Patient Use; Do Not Reuse
- Latex Free and Hypoallergenic
- Safe For Use in MRI